Original Dr. Berger

# KIRSCHKERNKISSEN

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#### Original Dr. Berger cherry pit cushion - The proven, versatile household remedy

Easy and safe application | Natural and gentle relief | Aromatic fragrance | Pleasant massage effect | Suitable for oven and microwave | Long-lasting, soothing heat

#### USER MANUAL

**Heating in the microwave:** Attention! Always put a cup of water in the microwave! Take the pillow out of the package. Place the cherry pit cushion in the middle of the microwave. Warm up at 600 watts for about 2 minutes. After warming up the cushion for approx. 30 sec. leave it in the microwave.

**Heating in the oven:** Take the pillow out of the package. Preheat the oven at top and bottom heat on 120° C and place the pillow on a cold rack covered with baking paper. Slide the rack into the oven on the middle shelf. Heat the cushion for approx. 20 minutes.

#### CARE INSTRUCTIONS

Allow the cushion to cool completely between uses. Store dry when not in use. Heat pads with natural fillers cannot be washed completely with their contents - danger of mould! If necessary, remove the filler beforehand!

#### SAFETY INSTRUCTIONS

Not suitable for allergy sufferers, people with weak immune systems or people who have no perception of heat or cold. Never overheat the pillow - danger of burns! Do not leave the pillow unattended when warming up. The application is intended to support your own therapy and does not replace a visit to the doctor. Do not use if the cover is damaged, discoloured or burned. Check always carefully the temperature with your hands after heating before use. Please keep these instructions for use!

Content: 1 cherry pit cushion | Size: 20 x 30 cm | Cover: 100 % Cotton | Filling: cherry pits | Colour: red, dotted

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